




PHOENIX 2

TALKING BATHROOM SCALE

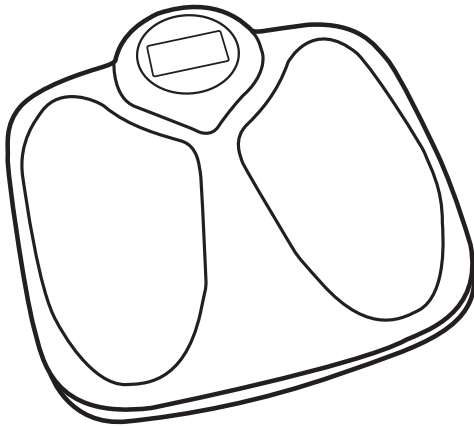


USER MANUAL

PHOENIX 2

TALKING BATHROOM SCALE



ENGLISH

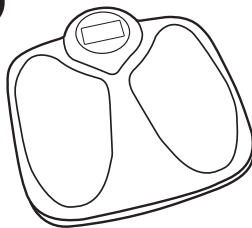

Capacity

180kg x 0.1kg

396lb x 0.2lb

28st : 5lb x 0.2lb

Thank you for purchasing the My Weigh® Phoenix 2 talking bathroom scale. Please read all operating instructions carefully before use. This electronic scale is a precision instrument. With normal care and proper treatment, it will provide years of reliable service.

The new My Weigh® Phoenix 2 Talking Scale is packed with features that will appeal to any user. The Phoenix 2 will audibly announce the measured weight, in addition to displaying it on the high contrast LCD. This feature has been especially important to users that are visually impaired or obese. This body scale has a maximum capacity of 396Lbs/180kg / 28 :5lb, and the ability to select from 4 different languages: English, Spanish, German, and French. The Phoenix 2 is the scale everyone is talking about...and listening to!

For more information please visit www.myweigh.com.

OPERATION INSTRUCTIONS

Batteries

Insert 4 x AAA into the battery compartment on the underside of the scale and make sure the polarity (+/-) is correct. If you do not intend to use this unit for a long period of time, remove the batteries before storage.

Selecting language

After showing "0.0", press and hold the [kg/lb/st] until you will hear "Hello" in the existing language. Press the [kg/lb/st] button again to select the language. You will hear "Hello" in the other languages as you toggle between the 4 options. To confirm the selection, simply leave the scale in the preferred language. The scale will save the setting and turn off.

Changing the Unit

Turn the Scale ON, "0.0" will show on the display" press the [kg/lb/st] button under the scale to select the desired unit lb, kg, st-lb.

Weighing

Use the scale on hard, flat, dry and secure surface only. This and most bathroom scales will not perform properly on thick-carpeted surfaces. Place the scale in an area free from high temperature or high humidity. To clean, use a dampened cloth and mild detergent. Avoid excessive impacts or vibrations to the unit.

1. Step onto the weighing platform. The scale will say “Hello”. Now step off the platform and wait until the scale is stable.
2. Once the scale is stable, it’ll say “It’s ready” and show “0.0”. Now you can step onto the weighing platform again. Your weight will be displayed and read out once the reading is steady.
3. You may step off of the platform and the reading will hold for 10 secs. After that it’ll say “Good-bye” and turn off automatically.

Switching on/off the talking function

Use the switch on the bottom of the platform to switch on/off the speaker function.

Auto off

The scale will shut down automatically after no operation is detected for about 10 seconds.

IMPORTANT NOTES

1. If the Scale ever malfunctions please try installing new batteries, sometimes low batteries or defective batteries may cause inaccurate readings or other errors
2. Do not attempt to disassemble the unit, otherwise the warranty will become void.
3. The unit is intended for home and personal use only (not legal for commerce).

| | | | |
|--------------------------|--|--------------|---------------|
| Capacity | 180kg x 0.1kg 396lb x 0.2lb 28st : 5lb x 0.2lb | Units | kg, lb, st-lb |
| Deviation | +/- 1% | | |
| Temperature range | Optimum 15-20°C | | |
| Power source | 4 x AAA | | |

